

revised 2017.10(ver.2)

## PG I /PG II calibrator

### PG I /PG II Calibrator Set

#### Intended Use

This product is for purpose of calibrating results in the quantitative determination of pepsinogen I and II in blood automated immunoassay. For in vitro diagnostic use only.

#### Precautions

HBs Ag :	Negative	(-)
HCV Ab :	Negative	(-)
HIV Ab :	Negative	(-)

The product must however be handled as potentially biohazardous material.

This product includes 0.09w/v% sodium azide as preservative.

After the measurement, it must be diluted with plenty of water and then drained away.

Storage : Dark and Cold (2~8 °C)

Lot No. : 129ACK

Exp. Date : 2025.05.

#### 【Calibrator value】

Level	Color (cap)	PG I concentration (ng/mL)	PG II concentration (ng/mL)
1	Colorless	0.0	0.0
2	Blue	12.5	6.3
3	Green	25.0	12.5
4	Yellow	50.0	25.0
5	Orange	100.0	50.0
6	Pink	200.0	100.0

manufactured by :

UMA Co., Ltd.

2-19-6 Yokosuka Matsudo City Chiba-Ken,

270-0031, Japan

phone: +81-47-710-4871

試験者	品質管理課長	製造管理者
		